



CLARUS™ SINGLE USE CLEANING BRUSHES

(ENGLISH IFU)

SINGLE USE FLEXIBLE SHAFT LUMEN
CLEANING BRUSHES
7125, 7162, 7163, 7164, 7165, 7166, 7243S, 7262

SINGLE USE RIGID SHAFT LUMEN CLEANING
BRUSHES
72008, 72010, 72608, 72610, 72612, 73212

Instructions for Use

2

Choose PriMed for the Perfect Swipe!

REFERENCES:

CSA/Z314.8-14;
PIDAC/NOV 2012;



PRIMED INSTRUMENTS INC.
1080 Tristar Drive, Unit 14
Mississauga, Ontario, L5T 1P1
Canada
PHONE: 905-565-0566
FAX: 905-565-0566
www.primedENDO.com

R_X ONLY

MD

INTENDED USE

These instruments are intended to be used as an aid in cleaning and disinfecting. Cleaning Brushes are used to scrub and loosen any debris present. Do not use these instruments for any purpose other than their intended use.

CONTRAINDICATIONS

For instruments with channel(s), do not use the cleaning brushes if the channel(s) are known to be occluded as this may result in damage to the channel. Never use excessive force when introducing or withdrawing the brush. Use of excessive force may damage the brush or the equipment being cleaned.

CAUTIONS

- Follow all cleaning instructions of the endoscope's Original Equipment Manufacturer.
- New, unused brushes are provided non-sterile. Do not use in a sterile or aseptic setting.
- Brushes should only be used in the decontamination area.
- Brushes should never be used ON or IN the human body.
- When using detergent and disinfection solution, always follow the manufacturer's recommendations regarding temperature, soaking time and concentration.

WARNINGS

- Patient debris and reprocessing chemicals are hazardous. Wear protective equipment at all times.
- After using, carefully check that no brush part have detached. Any part of the brush remaining in the channel after cleaning can drop into patient's body during a subsequent procedure. Detached part can be flush out with water or retrieved from inside the channel by passing a new brush or other appropriate size accessory through the channel or retrieve using a biopsy forceps.

REUSE WARNINGS

These brushes are for SINGLE USE ONLY. Do not clean, disinfect or sterilize these brushes prior to use. Doing so may damage the brush and/or the equipment being cleaned and/or impair its cleaning ability, which may cause infection to the patient. Any institution, practitioner or third party, who reprocesses, refurbishes, remanufactures, resterilizes, and or reuses these devices must bear full responsibility for their safety and effectiveness.

STORAGE

Store brushes in a manner that will reduce cross-contamination. Store cleaning brushes in original packaging at room temperature in a dry clean location away from direct sunlight.

EQUIPMENT REQUIRED

- Preparation, Inspection and Use: Single Use Brush.
- Cleaning: Ultrasonic cleaner, container, enzymatic detergent, cleaning brush, sponge & lint-free cloth.

PREPARATION

Before brushing, follow instruction manual provided by the original equipment manufacturer regarding preparation, inspection and cleaning.

INSPECTION

- Remove the brush from the package. Only handle the brush immediately before use.
- Inspect the brush before use for damages such as rust, sharp burr, frayed, bent, kinked, missing or detached bristles. This inspection is the responsibility of the user. Damaged bristles are ineffective in cleaning and could damage the equipment being cleaned.

2

- Do not repair damaged brushes. Use of repaired brushes may damage the instrument being cleaned. If damage is observed, discard and replace with a new brush.

USE

- Select the appropriate brush:** Choose a brush with the correct diameter and length for the equipment lumen being manually cleaned.
- Brush under water:** Always brush under water to minimize aerosolization of microorganisms.
- Advance the brush carefully:** Move the brush through the equipment channel or lumen in short strokes to avoid kinking. Do not rotate the brush inside the channel.
- Clean laparoscopic instrument shafts:** When cleaning a laparoscopic instrument's tubular shaft, ensure the cleaning brush enters and completely exits the shaft for proper cleaning.
- Remove gross soil:** Flush the device with water while using the brush through the channel until all soil is removed.
- Refer to manufacturer instructions:** For equipment-specific cleaning instructions, refer to the original equipment manufacturer's Instructions for Use (IFU).

PRODUCT DISPOSAL



Used cleaning brush may be a potential biohazard. Handle and dispose of in accordance with the hospital/clinic protocol and applicable regulations

SYMBOLS MEANING



Quantity of Units in Case.



Quantity of Units in Dispenser.



Product Number.



This product is not made with LATEX.



This Product is not made with PHTHALATE.



Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Visit us online at www.primedENDO.com for full list of symbol glossary including those in compliance with ISO 15223-1 standard.

NOTE: Contact PriMed Instruments if you have any questions of comments regarding the content of this document.

If two years have elapsed between the issued date for these instructions and product use, the user should contact Primed Instruments to determine if additional information is available.

CLARUS™ is a registered trademark of PriMed Instruments Inc.

Instructions for Use
W-7.5.3-W13A – CLARUS GENERAL DCB IFU

Revised & Approved & Effective

2025/02/12

© 2025 PriMed Instruments Inc.

3