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SINGLE-USE ENDOSCOPE CLEANING BRUSHES

CHANNEL CLEANING BRUSHES

9010, 9020, 9025, 9103, 9115/S, 9190, 9295, 9299

SUCTION WELL CLEANING BRUSHES

9990, 9295, 9299

SUCTION WELL AND VALVE CLEANING BRUSHES


9992, 99PTX2

CHANNEL CLEANING BRUSH KITS

9155, 9199

Choose PriMed for the Perfect Swipe!

DEVICE DESCRIPTION – Channel Cleaning Brush

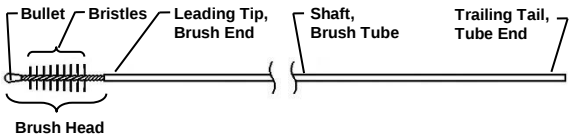
Suction Well Cleaning Brush

Brush Handle

Valve Cleaning Brush

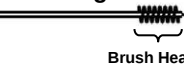R_XONLY

CE

INTENDED USE

These instruments are intended to be used as an aid in cleaning and disinfecting flexible endoscopes. Channel Cleaning Brushes are used to scrub the inside of the Endoscope Channel and loosen any debris present. Suction Well and Valve Cleaning Brushes are used to scrub the inside of Suction Well and Valve to loosen any debris present. Disposable Cleaning Brushes are single use devices and should be discarded after each cleaning of the endoscope. Do not use these instruments for any purpose other than their intended use.

SIGNAL SYMBOLS

WARNING	Indicates a potentially hazardous situation, which could result in death or serious injury.
CAUTION	Indicates a potentially hazardous situation, which may result in minor or moderate injury.
NOTE	Indicates additional helpful information.

CONTRAINDICATIONS

Do not use the cleaning brushes if the channel(s) are known to be occluded as this may result in damage to the channel. Never use excessive force when introducing or withdrawing the brush. Use of excessive force may damage the brush or the equipment being cleaned.

Follow all cleaning instructions of the endoscope's Original Equipment Manufacturer. Do not use a specific cleaning brush in a specific channel if use is not recommended by the endoscope's Original Equipment Manufacturer.

CAUTIONS

- Do not use Channel Cleaning Brushes for cytological tissue sampling or any other diagnostic or therapeutic purposes. Patient injury, cross-contamination, equipment damage may occur.
- Never coil brush into a diameter smaller than 20cm. The brush can be damaged if coiled too tightly.
- Only trained and qualified personnel should be given the responsibility to reprocess endoscopes.
- Ensure that the brush size is compatible with Endoscope Channel. See brush label for compatible channel size and length.
- When using detergent and disinfection solution, always follow the manufacturer's recommendations regarding temperature, soaking time and concentration.
- Do not attempt to insert the brush in any other way than specified in the manual provided by the endoscope manufacturer. Do not attempt to withdraw the brush from openings other than those specified in the manual provided by endoscope manufacturer. Doing so may cause brush to become lodged in the channel.
- Never use excessive force when introducing or withdrawing the brush. If a brush encounters significant resistance or blockage, do not apply undue force on the brush. Use of excessive force may damage the brush or the equipment being cleaned.
- Do not rotate, twist or torque the Channel Cleaning Brush while it is inside the channel. Doing so may damage the brush and/or the equipment being cleaned.
- Do not use damaged brushes. Always inspect brushes for damages before use. See Figures 2 -7. Damaged brushes may not adequately clean the endoscope channel, which may cause patient infection.
- Perform cleaning and disinfection with both the brush and the endoscope completely submerged in the detergent solution to prevent spattering.
- Do not put your face too close to the channel opening and do not withdraw the brush too quickly from the channel. Doing so may cause operator's infection or irritation from the splashes of patient blood/mucus and detergent solution.
- Withdraw the brush straight from the channel opening. Do not rub brush shaft, stem or bristle against the channel opening. Doing so may damage the brush or equipment being cleaned.
- Never transfer a contaminated brush between endoscopes; cross-contamination may occur resulting in patient infection.

WARNINGS

- Organic debris and reprocessing chemicals are hazardous. Before reprocessing the endoscope always wear personal protective equipment, such as eyewear, face mask, moisture-resistant clothing, and chemical-resistant gloves that fit properly and are long enough to cover exposed skin.
- After using, carefully check that no parts of the brush had fallen off. Any part of the brush remaining in the endoscope after cleaning can drop into patient's body during a subsequent procedure. Detached part can be retrieved from inside the endoscope channel by passing a new brush or other endoscope accessory through the channel.

REUSE WARNINGS

These brushes are for SINGLE USE ONLY. Do not clean, disinfect or sterilize these brushes prior to use. Doing so may damage the brush and/or the equipment being cleaned and/or impair its cleaning ability, which may cause infection to the patient.

STORAGE

Store cleaning brushes in original packaging at room temperature in a dry clean location away from direct sunlight.

EQUIPMENT REQUIRED

1. Preparation, Inspection and Use: Endoscope & cleaning brushes.
2. Cleaning: container, enzymatic detergent, high level disinfectant, cleaning brushes, sponge & lint-free cloth.

PREPARATION

1. Before brushing, follow instruction manual provided by the endoscope manufacturer regarding endoscope preparation, inspection and cleaning.

INSPECTION

1. Remove the brush from the package. When removing channel cleaning brush, carefully unwind the shaft. Only handle the brush immediately before use.
2. Inspect the brush before use for damages depicted in Figures 2-7. This inspection is the responsibility of the user. Worn out bristles are ineffective in cleaning, and damaged brushes may damage the endoscope. Although, Figures 2-7 depict the Channel Cleaning Brushes, the same inspection applies to Suction Well and Valve Cleaning Brushes.
3. If damage is observed, discard and replace with a new brush. Do not repair a damaged brush. Use of repaired brush may damage the endoscope.
4. Inspect the brush after use for any detached parts as depicted in Figure 7. If parts are detached or missing, make sure all detached parts are retrieved and properly discarded (see "DISPOSAL" instruction).

Figure 2 Missing bristles (Gap)

Figure 3 Bent (Set) bristles

Figure 4 Loose (Wild) bristles

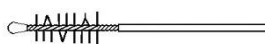
Figure 5 Permanent bristles adhesion

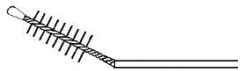
Figure 6 Damaged/kinked brush

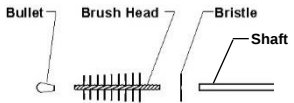
Figure 7 Brush sections that may detach

USE

CHANNEL CLEANING BRUSH

While the brush and the endoscope are submerged in cleaning solution, advance the leading tip of the brush (see Figure 8) into the endoscope channel opening until the leading brush head emerges out the other end of the channel. Use short strokes to avoid kinking. With the brush still submerged, scrub off all debris on the bristles before reversing its direction in the channel. Perform at least two passes through each channel or until no debris is observed upon inspection of brush.

VALVE CLEANING BRUSH

While the brush and the valve are submerged in cleaning solution, insert brush into the valve as shown in Figure 9. Gently clean the valve openings. With the brush still submerged, scrub off all debris on the bristles. Repeat cleaning valve until no debris is observed upon inspection of the brush.

SUCTION WELL CLEANING BRUSH

While the brush and the endoscope are submerged in cleaning solution, insert brush into the suction well or channel port until resistance is felt. See Figure 10. Rotate the inserted brush one full revolution and pull the brush out of the well. With the brush still submerged, scrub off all debris on the bristles. Repeat cleaning until no debris is observed upon inspection of the brush.

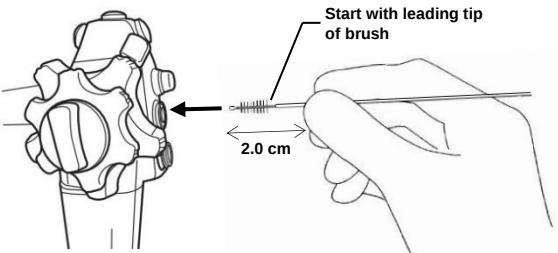
Figure 8 Cleaning endoscope channels

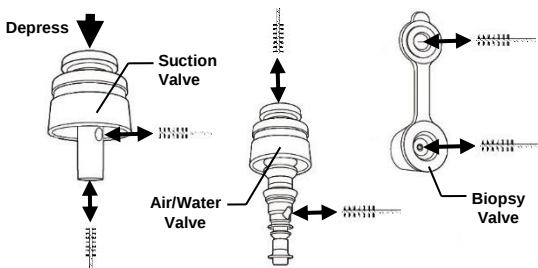
Figure 9 Cleaning accessory valves

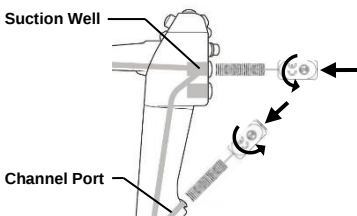
Figure 10 Cleaning suction well and channel ports

PRODUCT DISPOSAL

Used cleaning brush may be a potential biohazard. Handle and dispose of in accordance with the hospital/clinic protocol and applicable regulations.

SYMBOLS MEANING

Federal Law (USA) restricts this device to sale by or on the order of a physician.

Product Number.

Quantity of Units in Case.

This product is not made with Latex.

Quantity of Units in Dispenser.

This product is not made with Phthalate.

CE Mark.

Visit us online at www.primedENDO.com for full list of symbol glossary.

NOTE: Contact PriMed Instruments if you have any questions or comments regarding the content of this document.

If two years has elapsed between the issued date for these instructions and product use, the user should contact Primed Instruments to determine if additional information is available.

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© 2021 PriMed Instruments Inc.